

# NATURALLY SMOOTH

IN EVERY  
DIRECTION



patent  
pending

## *element* **DS** FOOT SYSTEM™

- > Smooth rollover and superior stability
- > Vertical shock and multi-axial rotation
- > Maintenance free design
- > Low profile design—6.5 in. build height
- > Rated for patients up to 275 lbs (125 kg)
- > Durable; meets the ISO-22675 test standard
- > 3 year warranty/30 day patient trial



**eMOTIS**  
A Fillauer Company

*Fillauer* LLC • Worldwide Distributor • 800.251.6398 • 423.624.0946 • [www.fillauer.com](http://www.fillauer.com)

# ELEMENT DS

from Fillauer LLC



The Element Foot's 3rd Element technology produces an incredibly smooth rollover. Adding the DuraShock increases the vertical shock absorption and adds to the multi-axial rotation. The elastomers and carbon components work together to absorb, store and return energy in vertical and horizontal planes. The result is a foot system that provides a high level function and requires a build height of only 6.5 inches.

## FEATURES & BENEFITS

- Smooth rollover and superior stability
- Vertical shock and 60° of axial rotation
- Innovative Micro-Slice technology for multi-axial performance
- Adjustable heel wedge for fine tuning heel stiffness
- Maintenance free design
- Low profile design—6.5 in. build height

## SPECIFICATIONS

- Patient weight: Up to 275 lbs. (125 kg)
- Foot weight: 22.3 oz. (632 g)
- Build height: 6.5 in. (16.5 cm)
- Functional level: K3–K4
- Durable; meets ISO-22675 standard

## L-CODES\*

- L5984** All endoskeletal lower extremity prosthesis, axial rotation unit.  
**L5987** All lower extremity prosthesis, shank foot system with vertical loading pylon

\*Suggested L-Codes are provided as a reference only, it is the responsibility of the practitioner to confirm this information.

## WARRANTY

- 3 years from the date of shipment to the practitioner
- Foot Shell: 6 months from the date of shipment to the practitioner

## SATISFACTION GUARANTEE

- 30 days from the date of shipment to the practitioner

## ORDERING INFORMATION

**105-10-XX-XXXX** Element DS

Element foot with DuraShock unit attached, spectra sock, heel bands, wedge elastomers and instructions. Foot selection is determined by foot size and patient weight. The foot model recommendations on the selection chart are based on average patients, engaged in normal activities. If your patient is much more active than average or is lifting heavy loads as part of their daily routine, then going up one weight category is advised.\*

		22 cm	23 cm	24 cm	25 cm	26 cm	27 cm	28 cm	29 cm	30 cm
100–119 lbs.	45–54 kg	22-A11R	23-A11R	24-A11R	25-A32R	26-A32R	NR	NR	NR	NR
120–139 lbs.	54–63 kg	22-A11R	23-A11R	24-A11R	25-A32R	26-A32B	27-A43B	28-A43B	NR	NR
140–159 lbs.	64–72 kg	22-A11R	23-A11R	24-A11R	25-A32B	26-A32B	27-A43B	28-A43K	29-C64K	30-C64K
160–179 lbs.	73–81 kg	22-A22R	23-A22B	24-A22B	25-A43B	26-A43B	27-A43K	28-A43K	29-C64K	30-C64K
180–199 lbs.	82–90 kg	22-A22B	23-A22B	24-A22B	25-B43B	26-B43K	27-B54K	28-B54K	29-C64K	30-C64K
200–224 lbs.	91–102 kg	22-A32B	23-A32K	24-A32K	25-B53K	26-B53K	27-B54K	28-B54K	29-D75K	30-D75K
225–249 lbs.	102–113 kg	NR	NR	NR	25-C54K	26-C54K	27-C65K	28-C65K	29-D75K	30-D75K
250–275 lbs.	113–125 kg	NR	NR	NR	25-C64K	26-C64K	27-C65K	28-C65K	29-D75K	30-D75K

## ACCESSORIES

**XX XX 13 CC 3** Micro Coated Vinyl Foot Shell  
 Example: 45 24 13 13 3 = Left, Size 24, Color 13

To order, select the side (left or right) and foot length (24–30 cm) from the chart below. Then, choose the color (CC 03, 09, or 13).

	22 cm	23 cm	24 cm	25 cm	26 cm	27 cm	28 cm	29 cm	30 cm
Left	45 22	45 23	45 24	45 25	45 26	45 27	45 28	45 29	45 30
Right	46 22	46 23	46 24	46 25	46 26	46 27	46 28	46 29	46 30

Custom colors are available as a special order with three weeks lead time. Please contact Fillauer Customer Service for more details.

### Spectra Sock

**192500** Spectra Sock\* (sold separately)  
 \*Recommended option when using foot shell.

