

Fillauer Tiny Titans™ Suspension Sleeves

REF 194100-194199

The physician or prosthetist should discuss indications, contraindications, washing instructions and application instructions with the patient and/or caregiver before use.

Indications

Suspension of prosthesis as prescribed by a prothetist.

Contraindications

Tiny Titans™ sleeves should not be worn while sleeping or by patients who are susceptible to dermatitis or have circulatory problems. If any rash, blisters, skin irritations, or circulatory problems do occur, the patient should discontinue use of the sleeve and consult their physician or prosthetist.

Washing Instructions

Tiny Titans™ sleeves should be washed regularly with cool soapy water. If Velcro® is used on the sleeve, it must be closed and locked before washing. DO NOT use excessive heat. Air dry or lay flat on a towel to dry. DO NOT use solvents or chlorine bleach.

Warranty

Defects in manufacturing or materials used will be covered for a period of (90) days from date of purchase. Items covered by warranty will either be replaced or repaired by the distributor. Sleeves must be returned CLEAN and DRY to the manufacturer. The manufacturer or distributor will not be liable for any damages or injuries rising from defective materials, workmanship, or problems covered under the Indications and Contraindications.

Fillauer, INC.®

2710 Amnicola Highway, Chattanooga, TN 37406, (423)624-0946, (800) 251-6398

European Representative, Basko Healthcare, Hapelsstraat 6, 1063 TE Amsterdam, The Netherlands



PM130/10-01