

## Fillauer Tiny Titans™ Knee Supports

The medical professional should discuss indications, contraindications, washing instructions and application instructions with the patient and/or caregiver before use.

### Indications

Soft tissue injuries, sprains, and strains where heat and support are prescribed by a medical professional. Commonly used to treat:

- Anterior Cruciate Instability
- Posterior Cruciate Instability
- Chondromalacia
- Lateral Collateral Ligament Instability
- Medial Collateral Ligament Instability
- Patellar Subluxation
- Patellar Tendonitis
- Patellar Dislocation
- Arthritis

### Contraindications

Tiny Titans™ supports should not be worn while sleeping or by patients who are susceptible to dermatitis or have circulatory problems. If any rash, blisters, skin irritations, or circulatory problems do occur, the patient should discontinue use of the support and consult their physician.


### Washing Instructions

Tiny Titans™ supports should be washed regularly with cool soapy water. If Velcro® is used on the support, it must be closed and locked before washing. DO NOT use excessive heat. Air dry or lay flat on a towel to dry. DO NOT use solvents or chlorine bleach.

### Warranty

Defects in manufacturing or materials used will be covered for a period of (90) days from date of purchase. Items covered by warranty will either be replaced or repaired by the distributor. Supports must be returned CLEAN and DRY to the manufacturer. The manufacturer or distributor will not be liable for any damages or injuries rising from defective materials, workmanship, or problems covered under the Indications and Contraindications.

### Pediatric Knee with Patella Cutout

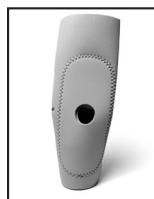

Front View

#### REF 194270-194319

- Larger end placed proximal
- Anterior opening with oval pad for patella
- Neoprene with nylon both sides

### Pediatric Knee with Enclosed Donut




Front View

#### REF 194320-194349

- Larger end placed proximal
- Anterior opening with tubular donut buttress for patella
- Medial and lateral spiral stays
- Neoprene with nylon both sides

### Pediatric Wrap-Around Knee with Patella Pull

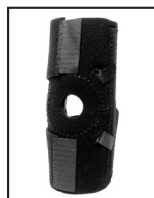

Front View

#### REF 194350-194355

- Plush outer fabric
- Anti-slip rubber inside
- Adjustable dynamic pull straps on patellar buttress
- Medial and lateral spiral stays

### Pediatric Knee with Velcro® Attached Buttress



Front View

#### REF 194200-194219

- Larger end placed proximal
- Anterior and posterior opening
- Reversible tubular horseshoe buttress for patella
- Neoprene with nylon both sides

### Pediatric Knee Mate with Hinges



Front View

#### REF 194220-194249

- Larger end placed proximal
- Large popliteal (posterior) opening
- Adjustable straps
- Hyperextension lock hinges
- Neoprene with nylon both sides

\*Not suggested for patella stabilization

### Pediatric Patella Knee with Universal Felt Horseshoe



Front View

#### REF 194250-194269

- Larger end placed proximal
- Adjustable felt horseshoe buttress
- Medial and lateral spiral stays
- Neoprene with nylon both sides

**Fillauer** LLC

Fillauer LLC • 2710 Amnicola Highway • Chattanooga, TN • (800) 251-6398 • Fax: (423) 629-7936  
European Representative • Centri® • Kung Hans Väg 2 • 192 68 Sollentuna • Sweden • +46 8 580 311 65 • Fax: +46 8 580 811 28

