

Fillauer Tiny Titans™ Elbow and Wrist Supports

The medical professional should discuss indications, contraindications, washing instructions and application instructions with the patient and/or caregiver before use.

Indications

Soft tissue injuries, sprains, and strains where heat and support are prescribed by a medical professional. Commonly used to treat:

- Sprains
- Strains
- Bursitis
- Tendonitis
- Arthritis

Contraindications

Tiny Titans™ supports should not be worn while sleeping or by patients who are susceptible to dermatitis or have circulatory problems. If any rash, blisters, skin irritations, or circulatory problems do occur, the patient should discontinue use of the support and consult their physician.

Washing Instructions

Tiny Titans™ supports should be washed regularly with cool soapy water. If Velcro® is used on the support, it must be closed and locked before washing. DO NOT use excessive heat. Air dry or lay flat on a towel to dry. DO NOT use solvents or chlorine bleach.

Warranty

Defects in manufacturing or materials used will be covered for a period of (90) days from date of purchase. Items covered by warranty will either be replaced or repaired by the distributor. Supports must be returned CLEAN and DRY to the manufacturer. The manufacturer or distributor will not be liable for any damages or injuries rising from defective materials, workmanship, or problems covered under the Indications and Contraindications.

Pediatric Action Wrist Support



REF 194370-194419

- Splint portion placed on palm side of hand
- Neoprene with nylon both sides
- Removable splint for washing
- Adjustable hook and loop closures

Pediatric Action Wrist with Thumb Support



REF 194420-194469

- Splint portion placed on palm side of hand
- Neoprene with nylon both sides
- Adjustable hook and loop closures

Pediatric Elbow Support



REF 194470-194509

- Larger end placed proximal
- Seam placed as shown in picture
- Neoprene with nylon both sides



Pediatric Hyperextension Elbow Support



REF 194520-194559

- Crisscross straps area placed antecubital (anterior). Closures are placed distal (as shown in picture)
- Neoprene with nylon both sides
- Two adjustable straps to control hyperextension

