


## MightyMite™ Slide Unit FC100160

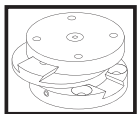

Compatible with standard pediatric four hole components.

### Ratings

- Weight ratings

60 kg (132 lbs) patient in office/clinic environment.

Lifetime is rated at 1 year based on normal use.

- Torque Ratings

Mounting Screws    M5 Flat Head Socket Cap  
6.9N-m (5.2 ft-lbs)

Set Screws            M4 Socket Head Cup Point  
2.1N-m (1.5 ft-lbs)

### Caution

- Not intended for definitive use.
- The practitioner should inspect the device for wear or damage before each use and after any accidents to confirm locking mechanisms are secure before allowing the patient to walk.
- When attaching a four hole device to the threaded four hole adapter make sure the screws are not too long. If the screws are too long they will lock the center plate adapter in place.

*Fillauer* LLC

2710 Amnicola Highway • Chattanooga, TN 37406 • Tel: 800.251.6398 • Fax: 423.629.7936 • [www.fillauer.com](http://www.fillauer.com)  
Centri® • Kung Hans Väg 2 • 192 68 Sollentuna, Sweden • Tel: +46 505 332 00 • Fax: 46 8 505 332 05 • [www.centri.se](http://www.centri.se)

