

FLAG

Force Limiting / Auto Grasp

A feature for Motion Control Hand and ETD

USER GUIDE FOR THE WEARER

What is FLAG?

FLAG is a special feature that allows you to limit your grip force, whenever you need to use more care when holding something delicate, like a child's hand, or a light bulb.

When to use FLAG

When you are learning to use your electric Hand or ETD, you may have situations when you do not yet "trust your own strength". You can feel safe that you won't crush what you are holding, by turning on **FLAG**. Use it for:

- Holding a child's hand
- Changing a diaper
- Holding a delicate glass . . . or a paper cup
- Holding an ice cream cone . . . or a peach
- Holding feathers delicately to tie a fishing fly

