

ProCover

Product Manual

Fillauer®

Intended Use

The Fillauer ProCover is made of durable and resilient polyurethane foam that enhances performance and protects the foot module. The split design makes the cover easy to install and remove for ease of maintenance and cleaning. The ProCover is intended for use with the AllPro, AllPro PM, or Wave Comfort 3.

Performance Characteristics

SkiTrek	
Compatible feet	AllPro, AllPro PM, AllPro XTS, Wave Comfort 3
Sizes	22 – 30 cm
Weight (26 cm)	9.1 oz. (259 g)
Material	Polyurethane foam
Activity level	K3 – K4

Storage and Handling


It is recommended that the ProCover is stored in a cool, clean, dry environment away from harsh chemicals (chlorine, acids, acetone, etc.).

Warranty

The ProCover is warranted for 9 months from date of installation.

Warranty is voided by alteration, misuse of product, or failure to comply with instructions.

Included Components

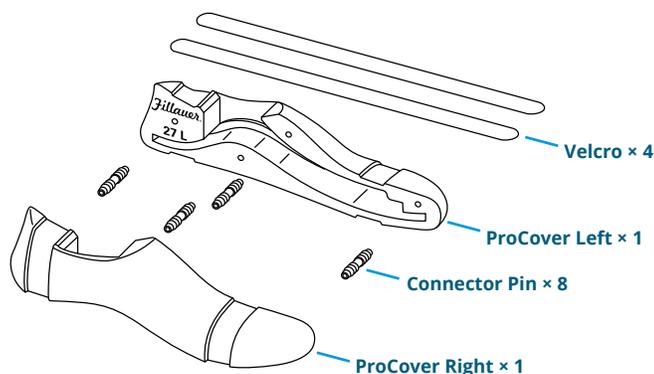

Replacement Components

Size	Component Replacement Kit Includes 4 Connector Pins and 2 Velcro® Straps
22 – 25	182-10-2300
26 – 30	182-10-2350

Warnings and Precautions



CAUTION: The ProCover is waterproof to 1 meter. However, if it is submerged, the foot and ProCover should be rinsed with fresh water and dried immediately to remove salt, chlorine, or debris.



CAUTION: The ProCover is designed to provide maximum performance of the foot. The life of the ProCover will depend on level of activity and degree to which it is protected from wear and damage with socks and shoes. Socks and shoes should be worn at all times and should be allowed to dry fully after exposure to water to prevent damage to the shell.



CAUTION: Patients should inspect the ProCover daily for signs of cracks or holes and for the presence of sand or other debris. If the ProCover shows signs of failure, it should be replaced as soon as possible to prevent damage to the carbon fiber and soling materials. If debris is present, the foot and ProCover should be rinsed and allowed to dry fully.



CAUTION: The ProCover may also be cleaned with a soft cloth and a soap and water solution or with rubbing alcohol (70%). **Do not use acetone. It will damage the ProCover.**

Qualified Provider


Attachment, adjustment, alignment, and delivery of this device must be performed by or under the direct supervision of a qualified prosthetist. Unless stated in this manual, any such activities should not be attempted by the user and will potentially void the device warranty.

Specifications and Preparations Before Use (Risk Management for Installation and Calibration)


Attention: Deviating from the installation instructions or modifying the ProCover in any way will void any product warranty and could lead to product failure and injury to the patient.

ProCover Installation

1. Insert the foot into one side of the ProCover and press down firmly so the foot is fully seated in the internal cavity.
2. Install the other side by aligning the four connector pins and pressing firmly in to place. It is normal to have a slight gap between the two halves but this can be minimized by firmly squeezing the two sides together.

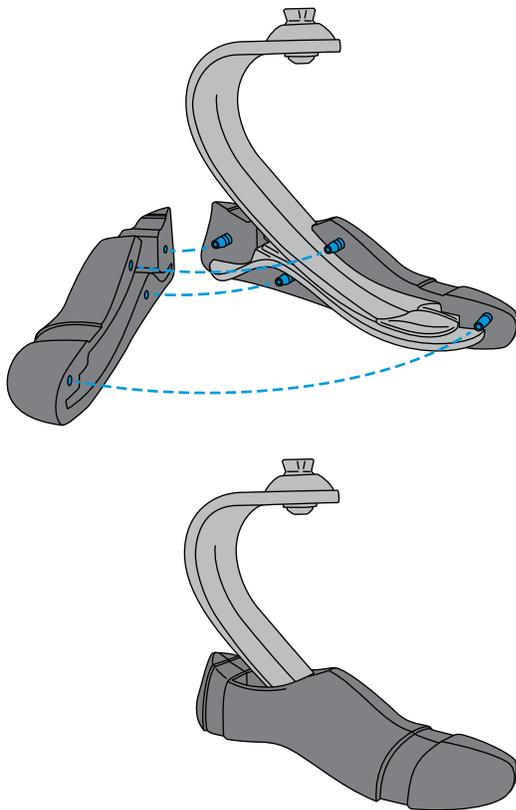

3. Install the two Velcro® straps by tightly wrapping them around the toe and heel channels with the soft sides of the straps facing out.

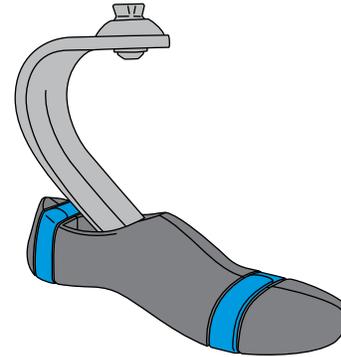

4. The cover can be easily removed to aid cleaning the foot and keep the foot in good working order.

Compatibility

The ProCover is appropriate for use with the adult AllPro DM, AllPro PM, AllPro XTS, Wave Comfort prosthetic feet. The fit of other manufacturers feet cannot be guaranteed.

Disposal / Waste Handling

The product must be disposed of in accordance with applicable local laws and regulations. If the product has been exposed to bacteria or other infectious agents, it must be disposed of in accordance with applicable laws and regulations for the handling of contaminated material.

Fillauer

www.fillauer.com



Fillauer

2710 Amnicola Highway
Chattanooga, TN 37406
423.624.0946



Fillauer Europe

Kung Hans väg 2
192 68 Sollentuna, Sweden
+46 (0)8 505 332 00



© 2023 Fillauer LLC
PM350/10-29-18/09-18-23/Rev.2