




# Posterior Mount System

*Fillauer*®

# Fit

# Transfer

# Finish

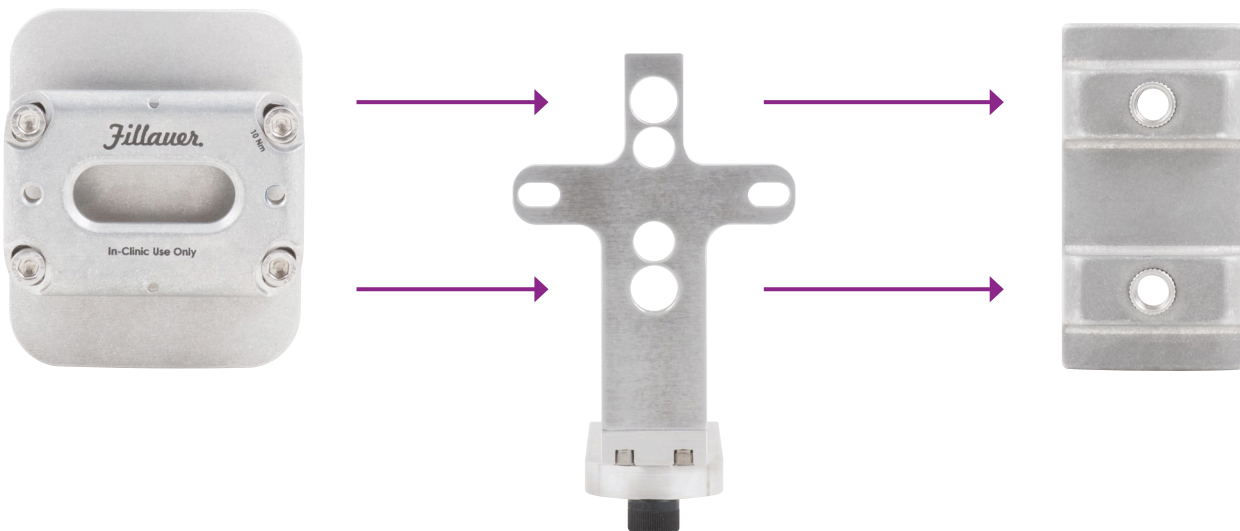

## Posterior Mount System

The **Posterior Mount System** allows for the quick, easy, and accurate alignment and transfer of Fillauer posterior mounted adult and pediatric prosthetic feet.

There are three basic components to help take a patient from bench alignment all the way to lamination. The **Adjustable Alignment Bracket** tool (pediatric or adult) allows a clinician to accurately align a prosthetic foot with simple height adjustment, medial-lateral adjustment, and frontal plane angulation.

After optimal alignment is obtained, the **Posterior Mount Transfer Fixture** allows the adjustable bracket tool to be replaced with a **Posterior Mounting Bracket** (pediatric or adult) for an adjustable, easy to remove, definitive option. The foot may also be directly laminated to the socket while maintaining perfect alignment

**The complete system will significantly improve fittings, save time, and assist to provide the best fit possible for patients.**

*Fillauer*®

# Adjustable Alignment Bracket

The bracket is for in-clinic fitting only — it is not a permanent mounting solution, but rather a fitting tool. The bracket is compatible with the Formula and AllPro Posterior Mount feet.



Adult

Pediatric

## Adjustable Alignment Bracket

Compatible Foot	Patient Weight	Weight	Dimensions	Socket Offset Distance	Part Number
AllPro PM	330 lbs. (150 kg)	4.8 oz. (136 g)	4.0 × 3.3 × 1.8 in. (100 × 83 × 46.4 mm)	0.16 in. (4 mm)	<b>180-10-4000</b>
Formula™	330 lbs. (150 kg)	4.8 oz. (136 g)	4.0 × 3.3 × 1.8 in. (100 × 83 × 46.4 mm)	0.16 in. (4 mm)	<b>180-10-4000</b>
Pediatric Formula™	132 lbs. (60 kg)	4.8 oz. (136 g)	3.7 × 2.9 × 1.0 in. (95 × 73 × 25 mm)	0.16 in. (4 mm)	<b>180-10-5000</b>

# Posterior Mount Transfer Fixture

The Posterior Mount Transfer Fixture attaches to the Hosmer VFJ and allows for quick adjustment of rotation and linear adjustment of alignment. It can be used with all Fillauer Posterior Mounting Brackets and Adjustable Alignment Brackets (Adult and Pediatric versions).




## Posterior Mount Transfer Fixture

Description	Product Number
Posterior Mount Transfer Fixture	<b>52090</b>
Vertical Fabricating Fixture, VFJ-100	<b>52056</b>

# Posterior Mounting Bracket

Once correct alignment has been determined using the Adjustable Alignment Bracket, the foot can be mounted using the Posterior Mounting Bracket.

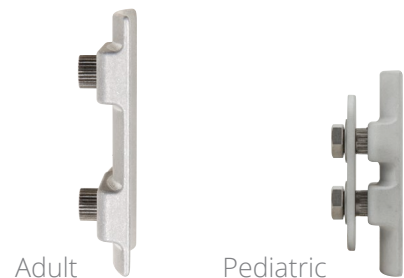

Adult

Pediatric

## Posterior Mounting Bracket

Compatible Foot	Patient Weight	Plate Thickness	Weight	Product Number
Aeris Activity	330 lbs. (150 kg)	0.3 in. (8.6 mm)	3.1 oz. (87 g)	<b>180-10-2000</b>
Running Blade	330 lbs. (150 kg)	0.3 in. (8.6 mm)	3.1 oz. (87 g)	<b>180-10-2010</b>
AllPro PM	330 lbs. (150 kg)	0.3 in. (8.6 mm)	3.1 oz. (87 g)	<b>180-10-2010</b>
Formula™	330 lbs. (150 kg)	0.3 in. (8.6 mm)	3.1 oz. (87 g)	<b>180-10-2010</b>
Pediatric Formula™	125 lbs. (57 kg)	0.3 in. (8.6 mm)	3.4 oz. (95 g)	<b>180-10-3000</b>

*Fillauer*<sup>®</sup>

[www.fillauer.com](http://www.fillauer.com)

**Fillauer LLC**

2710 Amnicola Highway  
Chattanooga, TN 37406  
423.624.0946  
[info@fillauer.com](mailto:info@fillauer.com)

© 2019 Fillauer LLC  
AD390/04-11-19