

Dynamic Walk

Care & Use

To reshape, spot heat the carbon fiber with boiling water or a heat gun up to 250 °F (121 °C) (higher temperatures will cause delamination). **Do not heat or reshape the PEEK rods or the carbon fiber PEEK attachments.** Attach padding kit. Select a laced shoe with a 5/8 in. (16 mm) heel and removable insole, and check to make sure that the patient's footwear does not place pressure on the PEEK rods. Excessive pressure on the rods may cause the Kevlar protective sheath to tear prematurely and damage the rods. Make sure to place the foot plate under the insole or foot orthosis in the shoe. For best results, use a custom foot orthosis.

Warranty

1 year warranty*

*Demo units are not warranted and are not for resale.

Standard Padding Kits

700100220	Padding Kit, Small
700100240	Padding Kit, Medium
700100260	Padding Kit, Large

Single Side Padding Kits

700100321	Padding Kit, Small, Left Lateral or Right Medial
700100322	Padding Kit, Small, Right Lateral or Left Medial
700100341	Padding Kit, Medium, Left Lateral or Right Medial
700100342	Padding Kit, Medium, Right Lateral or Left Medial
700100361	Padding Kit, Large, Left Lateral or Right Medial
700100362	Padding Kit, Large, Right Lateral or Left Medial

Swedish Patent No. 531 069

U.S. Patent No. 7,766,851

Fillauer®

www.fillauer.com

Fillauer LLC

2710 Amnicola Highway
Chattanooga, TN 37406
423.624.0946
Fax 423.629.7936



© 2016 Fillauer LLC
PM231 09-08/05-15-19