

Absolute  
Sleeve ▾

Absolute  
Liner ▾

*Fillauer*®

# ABSOLUTE SLEEVE

---

Absolute Sleeve are exceptionally comfortable and durable which combines intimate fit with the latest in sleeve technology. Extra cushioning combines superior comfort with protection for the wearer, while our soft, low friction surface allows it to slide easily against clothing.

Medical grade mineral oil is infused in Absolute Sleeve's gel polymer to make sure that skin is cared for with our unique, enriched treatments. The gel is ISO-tested and FDA approved and extra soft for comfort and improved wearability.

Durability is enhanced by using the highest quality of elastomers and materials.

By being thermoformable, Absolute Sleeve can be custom adapted to the precise body shape of each individual patient.

---

## Care and Maintenance

Cleani-Stump is ideal for cleaning Absolute Sleeve. A mild soap solution or alcohol can also be used. Absolute Sleeve should be rinsed thoroughly after cleaning to prevent any residue from being retained as this could cause skin irritation.

CAUTION: Using acetone or similar chemical solvents will damage Absolute Sleeve and void the warranty.

---

## Features Absolute Sleeve

- Available in black or beige
- Available in 8 sizes
- Thermoformable for custom fit
- Infused medical grade oil for skin protection
- FDA approved elastomer
- Greater heat transfer than silicone
- Extra Soft Gel; Shore A - 00
- 3 month warranty

US Patent 5,830,237; 6,964,688; 7,291,182  
European Patent 0762857; 1618858  
Canadian Patent 2,190,764

Licensed from Ohio Willow Wood

# ABSOLUTE SLEEVE

The Absolute Preflexed- & Straight Sleeve comes in black and beige. The fabric has a high elasticity giving a better fit and a low friction surface allowing it to slide easily against clothing. The gel is FDA approved, less than 3mm thick and Extra Soft which adheres comfortably to the skin with it's tacky surface.

The preflexed sleeve is reinforced on the inside of the knee to ensure a longer life.

Absolute Preflexed Suspension Sleeve 3mm - Extra Soft				
Size	Distal Circumference	Proximal Circumference*	Black	Beige
Small	21 - 25 cm	26 - 31 cm	803 461 314	803 471 314
Medium	26 - 32 cm	31 - 38 cm	803 461 318	803 471 318
Medium+	30 - 36 cm	38 - 44 cm	803 461 322	803 471 322
Large	32 - 40 cm	44 - 50 cm	803 461 326	803 471 326
Large+	34 - 42 cm	50 - 60 cm	803 461 332	803 471 332
X-Large	40 - 49 cm	60 - 70 cm	803 461 340	803 471 340

\*NOTE: The proximal circumference is measured 15 cm above the mid-patella tendon.



CE

Absolute Straight Suspension Sleeve 3 mm - Extra Soft				
Size	Distal Circumference	Proximal Circumference*	Black	Beige
X-Small	10 - 17 cm	16 - 24 cm	803 361 301	803 371 301
Small	15 - 22 cm	23 - 34 cm	803 361 302	803 371 302
Medium	21 - 30 cm	29 - 43 cm	803 361 303	803 371 303
Medium+	24 - 35 cm	33 - 47 cm	803 361 304	803 371 304
Large	28 - 42 cm	36 - 51 cm	803 361 305	803 371 305
Large+	31 - 46 cm	39 - 55 cm	803 361 306	803 371 306
X-Large	34 - 50 cm	42 - 60 cm	803 361 307	803 371 307
XX-Large	36 - 55 cm	44 - 70 cm	803 361 308	803 371 308

\*NOTE: The proximal circumference is measured 15 cm above the mid-patella tendon.



CE

# ABSOLUTE LINER

---

Absolute Liner are exceptionally comfortable and durable which combines intimate fit with the latest in sleeve and liner technology. Extra cushioning combines superior comfort with protection for the wearer, while our soft, low friction surface allows it to slide easily against clothing.

Medical grade mineral oil is infused in Absolute Liner's gel polymer to make sure that skin is cared for with our unique, enriched treatments. The gel is ISO-tested and FDA approved and extra soft for comfort and improved wearability.

Durability is enhanced by using the highest quality of elastomers and materials.

## Thermoforming

Preparation - Use a dry positive cast without sharp edges. Don a sock on to the cast in order to get an air channel and prevent "airlock" that can damage the gel in the Absolute Liner.

1. - Don the Absolute Liner on the room temperature cast
2. - Put the oven temperature to 80°C. Avoid bumping the liner against the oven walls as this can damage the gel.
3. - After 50 minutes take out the cast with the liner on and leave it to cool down at room temperature. DO NOT cool down the liner under cold water as this can damage the gel.

By being thermoformable, Absolute Liner can be custom adapted to the precise body shape of each individual patient. For casts with severe concave areas, e.g. "pear shaped" limbs, you might have to vacuum form the liner or order a custom made liner according to your specifications.

---

## Care and Maintenance

Cleani-Stump is ideal for cleaning Absolute Liner. A mild soap solution or alcohol can also be used. Absolute Liner should be rinsed thoroughly after cleaning to prevent any residue from being retained as this could cause skin irritation.

CAUTION: Using acetone or similar chemical solvents will damage Absolute Liner and void the warranty.

---

## Features Absolute Liner

- Available in black or beige
- Available in 7 sizes
- Thermoformable for custom fit
- Infused medical grade oil for skin protection
- FDA approved elastomer
- Greater heat transfer than silicone
- Extra Soft Gel; Shore A - 00
- Can be custom made
- 6 month warranty for liner with fabric
- 12 month warranty for liner if used in pairs

US Patent 5,830,237; 6,964,688; 7,291,182  
European Patent 0762857; 1618858  
Canadian Patent 2,190,764


Licensed from Ohio Willow Wood



# LOCKING LINER - THIN BOTTOM

All standard locking liners come with a 10 cm distal matrix.

For ordering this liner with full matrix you replace the third number with a nine, eg, a small uniform liner with product number 805 771 302 will become 809 771 302 as a full matrix liner.

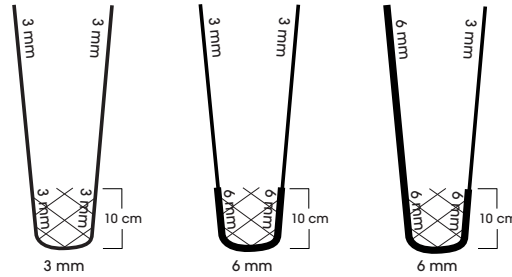

LOCKING LINER beige - thin bottom			
SIZE	Uniform 3 mm	Tapered 6 mm	Contoured 6 + 3 mm
X-SMALL	805 771 301	805 971 601	805 871 601
SMALL	805 771 302	805 971 602	805 871 602
MEDIUM	805 771 303	805 971 603	805 871 603
MEDIUM+	805 771 304	805 971 604	805 871 604
LARGE	805 771 305	805 971 605	805 871 605
LARGE+	805 771 307	805 971 607	805 871 607
X-LARGE	805 771 308	805 971 608	805 871 608
Distal thickness	3 mm	6 mm	6 mm


LOCKING LINER black - thin bottom			
SIZE	Uniform 3 mm	Tapered 6 mm	Contoured 6 + 3 mm
X-SMALL	805 761 301	805 961 601	805 861 601
SMALL	805 761 302	805 961 602	805 861 602
MEDIUM	805 761 303	805 961 603	805 861 603
MEDIUM+	805 761 304	805 961 604	805 861 604
LARGE	805 761 305	805 961 605	805 861 605
LARGE+	805 761 307	805 961 607	805 861 607
X-LARGE	805 761 308	805 961 608	805 861 608
Distal thickness	3 mm	6 mm	6 mm

SIZE CHART		
SIZE	Distal Circumference 5 cm from distal end	Proximal Circumference 40 cm from distal end
X-SMALL	14 - 20 cm	23 - 33 cm
SMALL	18 - 25 cm	25 - 36 cm
MEDIUM	22 - 31 cm	30 - 43 cm
MEDIUM+	23 - 33 cm	36 - 52 cm
LARGE	26 - 37 cm	34 - 49 cm
LARGE+	32 - 46 cm	41 - 59 cm
X-LARGE	40 - 58 cm	47 - 68 cm

# LOCKING LINER - THICK BOTTOM

All standard locking liners come with a 10 cm distal matrix.

For ordering this liner with full matrix you replace the third number with a seven, eg, a small uniform liner with product number 802 771 302 will become 807 771 302 as a full matrix liner.

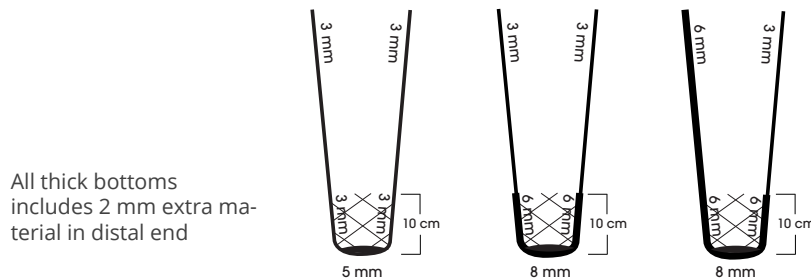

All thick bottoms includes 2 mm extra material in distal end


LOCKING LINER beige - thick bottom			
SIZE	Uniform 3 mm	Tapered 6 mm	Contoured 6 + 3 mm
SMALL	802 771 302	802 971 602	802 871 602
MEDIUM	802 771 303	802 971 603	802 871 603
MEDIUM+	802 771 304	802 971 604	802 871 604
LARGE	802 771 305	802 971 605	802 871 605
LARGE+	802 771 307	802 971 607	802 871 607
X-LARGE	802 771 308	802 971 608	802 871 608
Distal thickness	5 mm	8 mm	8 mm

LOCKING LINER black - thick bottom			
SIZE	Uniform 3 mm	Tapered 6 mm	Contoured 6 + 3 mm
SMALL	802 761 302	802 961 602	802 861 602
MEDIUM	802 761 303	802 961 603	802 861 603
MEDIUM+	802 761 304	802 961 604	802 861 604
LARGE	802 761 305	802 961 605	802 861 605
LARGE+	802 761 307	802 961 607	802 861 607
X-LARGE	802 761 308	802 961 608	802 861 608
Distal thickness	5 mm	8 mm	8 mm

SIZE CHART		
SIZE	Distal Circumference 5 cm from distal end	Proximal Circumference 40 cm from distal end
SMALL	18 - 25 cm	25 - 36 cm
MEDIUM	22 - 31 cm	30 - 43 cm
MEDIUM+	23 - 33 cm	36 - 52 cm
LARGE	26 - 37 cm	34 - 49 cm
LARGE+	32 - 46 cm	41 - 59 cm
X-LARGE	40 - 58 cm	47 - 68 cm

# CUSHION LINER - THIN BOTTOM

For ordering this liner with full matrix you replace the third number with a eight, eg, a x-small uniform liner with product number 804 771 301 will become 808 771 301 as a full matrix liner.

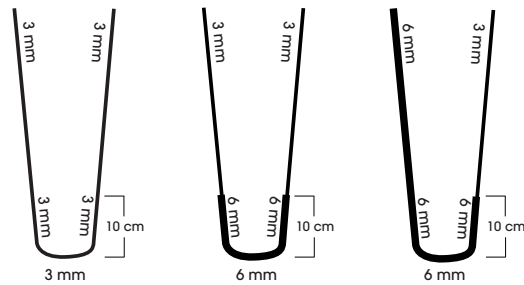

CUSHION LINER beige - thin bottom			
SIZE	Uniform 3 mm	Tapered 6 mm	Contoured 6 + 3 mm
X-SMALL	804 771 301	804 971 601	804 871 601
SMALL	804 771 302	804 971 602	804 871 602
MEDIUM	804 771 303	804 971 603	804 871 603
MEDIUM+	804 771 304	804 971 604	804 871 604
LARGE	804 771 305	804 971 605	804 871 605
LARGE+	804 771 307	804 971 607	804 871 607
X-LARGE	804 771 308	804 971 608	804 871 608
Distal thickness	3 mm	6 mm	6 mm

CUSHION LINER black - thin bottom			
SIZE	Uniform 3 mm	Tapered 6 mm	Contoured 6 + 3 mm
X-SMALL	804 761 301	804 961 601	804 861 601
SMALL	804 761 302	804 961 602	804 861 602
MEDIUM	804 761 303	804 961 603	804 861 603
MEDIUM+	804 761 304	804 961 604	804 861 604
LARGE	804 761 305	804 961 605	804 861 605
LARGE+	804 761 307	804 961 607	804 861 607
X-LARGE	804 761 308	804 961 608	804 861 608
Distal thickness	3 mm	6 mm	6 mm


SIZE CHART		
SIZE	Distal Circumference 5 cm from distal end	Proximal Circumference 40 cm from distal end
X-SMALL	14 - 20 cm	23 - 33 cm
SMALL	18 - 25 cm	25 - 36 cm
MEDIUM	22 - 31 cm	30 - 43 cm
MEDIUM+	23 - 33 cm	36 - 52 cm
LARGE	26 - 37 cm	34 - 49 cm
LARGE+	32 - 46 cm	41 - 59 cm
X-LARGE	40 - 58 cm	47 - 68 cm



# CUSHION LINER - THICK BOTTOM

For ordering this liner with full matrix you replace the third number with a six, eg, a x-small uniform liner with product number 801 771 301 will become 806 771 301 as a full matrix liner.

All thick bottoms includes 2 mm extra material in distal end

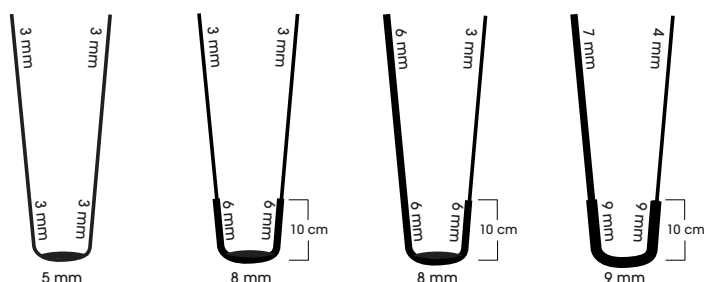

CUSHION LINER beige - thick bottom				
SIZE	Uniform 3 mm	Tapered 6 mm	Contoured 6 + 3 mm	Contoured 9 mm
X-SMALL	801 771 301	801 971 601	801 871 601	804 871 901
SMALL	801 771 302	801 971 602	801 871 602	804 871 902
MEDIUM	801 771 303	801 971 603	801 871 603	804 871 903
MEDIUM+	801 771 304	801 971 604	801 871 604	804 871 904
LARGE	801 771 305	801 971 605	801 871 605	804 871 905
LARGE+	801 771 307	801 971 607	801 871 607	NA
X-LARGE	801 771 308	801 971 608	801 871 608	NA
Distal thickness	5 mm	8 mm	8 mm	9 mm

CUSHION LINER black - thick bottom				
SIZE	Uniform 3 mm	Tapered 6 mm	Contoured 6 + 3 mm	Contoured 9 mm
X-SMALL	801 761 301	801 961 601	801 861 601	804 861 901
SMALL	801 761 302	801 961 602	801 861 602	804 861 902
MEDIUM	801 761 303	801 961 603	801 861 603	804 861 903
MEDIUM+	801 761 304	801 961 604	801 861 604	804 861 904
LARGE	801 761 305	801 961 605	801 861 605	804 861 905
LARGE+	801 761 307	801 961 607	801 861 607	NA
X-LARGE	801 761 308	801 961 608	801 861 608	NA
Distal thickness	5 mm	8 mm	8 mm	9 mm

SIZE CHART		
SIZE	Distal Circumference 5 cm from distal end	Proximal Circumference 40 cm from distal end
X-SMALL	14 - 20 cm	23 - 33 cm
SMALL	18 - 25 cm	25 - 36 cm
MEDIUM	22 - 31 cm	30 - 43 cm
MEDIUM+	23 - 33 cm	36 - 52 cm
LARGE	26 - 37 cm	34 - 49 cm
LARGE+	32 - 46 cm	41 - 59 cm
X-LARGE	40 - 58 cm	47 - 68 cm

# Orderform Custom Made Liner



Kung Hans väg 2  
192 68 Sollentuna  
Sweden

Phone: +46 8 505 332 00  
E-mail: fillauer@fillauer.com

Customer:

Fillauer's PO#:

Patient name:


Ordered by:

Phone:

Ordering information		
First order:	<input type="checkbox"/>	
Repeat order:	<input type="checkbox"/>	
Quantity	Type of liner	Product no.
	Cushion liner, black	801 100 301
	Cushion liner, beige	801 100 401
	Locking liner, black	802 100 301
	Locking liner, beige	802 100 401
Option:		
	3 mm Uniform (picture 1)	
	6 mm Tapered (picture 2)	
	3+6 mm Contoured (picture 3)	
	9 mm Contoured (only available in cushion, thick bottom, picture 4)	
	Thick bottom	
	Thin bottom	
	Full Matrix, recommended for AK prosthetic users	

## IMPORTANT Information

- The plaster model needs to be a positive model.
- Make sure that the model has the same length as the final liner.
- The model should be reduced approx. 4 - 6% on circumferential measurements.
- Please state if and where you need extra cushion or other custom solutions to your liner.

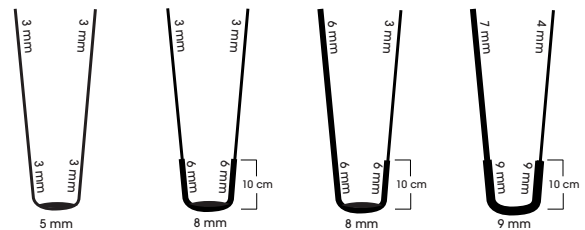

Pic.1

Pic.2

Pic.3

Pic.4

## Additional information:



Real Technology - Real People

*Fillauer*<sup>®</sup>

[www.fillauer.se](http://www.fillauer.se)

Kung Hans väg 2  
192 68 Sollentuna, Sweden  
Phone: +46 (0)8 505 332 00  
E-mail: [fillauer@fillauer.com](mailto:fillauer@fillauer.com)

PBE184 - AbsoluteSleeve/AbsoluteLiner  
2020-11-05