

## Manual EndoArm-Friction Flexion Rotation wrist

Assemble unit with a small amount of Loctite 242 or similar, applied to the M5X20 thread to ensure the chosen friction setting of flexion/ extension resistance. Tighten the M3X16 bolt to ensure that the tube is secured in place. Do not exceed 2Nm torque.

The M12X1,5 mm threaded stud is secured preferably to the chosen hand with a suitable adhesive.

*Fillauer*<sup>®</sup>

Kung Hans väg 2  
192 68 Sollentuna, Sweden  
Phone: +46 (0)8 505 332 00  
Fax: +46 (0)8 505 332 05  
E-mail: [fillauer@fillauer.eu](mailto:fillauer@fillauer.eu)  
[www.fillauer.eu](http://www.fillauer.eu)