




Quick Start Guide for Clinicians


TASKA Medium (7¾")
& Large (8¾") Hand V1.1

 **TASKA™**


1. Check

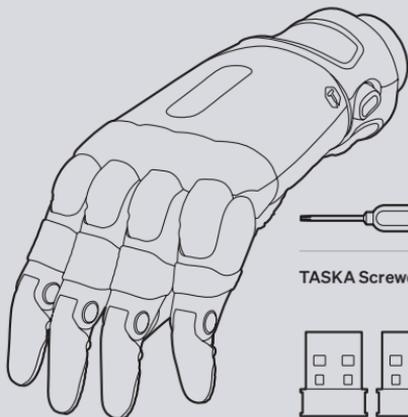


TASKA Hand*

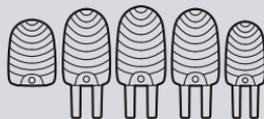

1 x Thumb & 4 x Finger Grip Tips

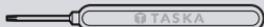

TASKA Screwdriver

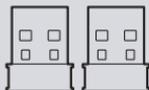

2 x Bluetooth Adaptors




TASKA Battery



2 x Top Faceplates



Lamination Dummy Blocks

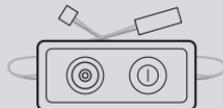


Power Switch


 do more

TASKA™

* Image shows Quick Disconnect Wrist fitting.

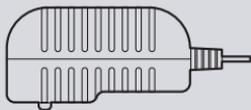

Mains Charger

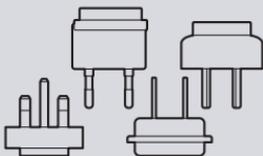

Universal Plug Kit




Car Charger

Low Profile Wrist only




2 x O-Rings

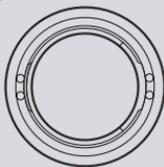

Lamination Ring



Masking Plug

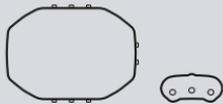

Switch Blocks



Wrist Lubricant



4 x Wrist Set Screws

 do more

TASKA™

Items shown are not to scale.

2. Download

HandCal for clinicians & users

To receive your download link for the TASKA HandCal clinician software, please email:

support@taskaprosthetics.com

For the TASKA HandCal user software, please enter the link below:


taskaprosthetics.com/support

For assistance on using the HandCal software, comprehensive guides are available for download on our website.

HandCal app

A free app is also available for mobile which allows configuration of button and EMG grip sets as well as troubleshooting sensor issues.

 do more

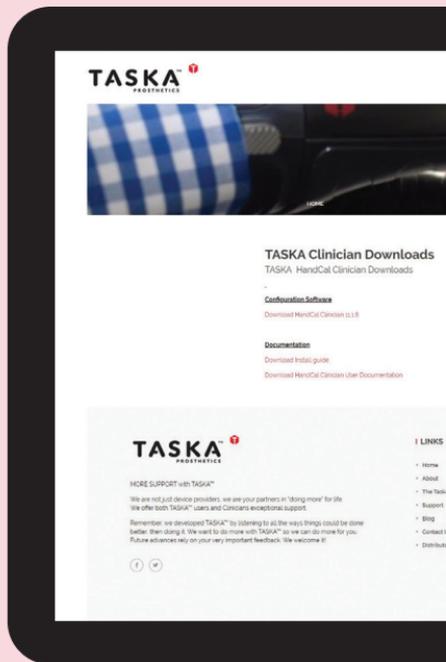

TASKA™

3. Control

Fault light

If this light stays on, there is an internal error. You won't be able to move the hand until you turn it off and on again.

Run the HandCal software to check for problems with your hand. Contact your clinician if this happens regularly.

EMG functions enabled

Light indicates when all EMG grip triggers or all inputs/EMG sensors are turned on and off.

Button grip set light

Light flashes momentarily when the first grip in the button group is selected.

Home grip set

Press this button to toggle between flexi-tool and general grasp. To select pincer precision grip, press the button for one second.

Button group triggers

Press this button to cycle through your predefined button group grips.

Press for one second to turn Bluetooth on or off.

Battery light

Light flashes every 10 seconds when the battery reaches 10%. When the battery is flat, the light will stay on and you won't be able to move the hand.

Bluetooth light

Illuminated when enabled.

EMG functions

Change this button's function using the Handcal software to turn off EMG sensors or turn off EMG triggers.

Turn sound on/off with one second hold.



4. Attach

Connecting the hand


If the TASKA Hand has been supplied with a Quick Disconnect Wrist, it is easy to attach and remove the hand.

To attach the hand:

1. Align the wrist connection of the hand with the socket.
2. Push them together. You will hear a click when the hand has locked to the socket.

To remove the hand:

1. Rotate the hand and socket in opposite directions and pull the hand gently away from the socket.
2. When the rotation has reached the correct point, the hand will release from the socket.

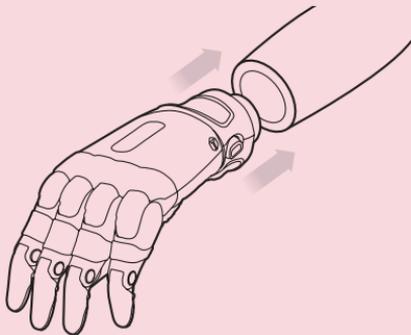

Connecting the Quick Disconnect Wrist.

Low Profile Wrist

For all TASKA Hands that have been supplied with a Low Profile Wrist, please refer to the Low Profile Installation Guide for instructions for use.


5. Power

Turning the hand on and off

There is a button on the socket that turns the TASKA Hand on and off.

Before turning on the hand, make sure there is nothing to stop the fingers and thumb from opening fully; this will help to correctly set the position sensors while the hand turns on.

To turn on the hand, press the button once. As the hand turns on, two of the lights on the back of the hand will flash briefly and the fingers will open when the hand is ready.

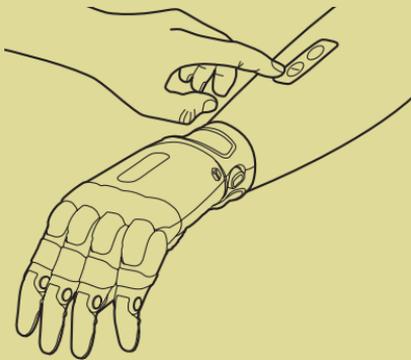

Turning the TASKA Hand on and off.



If using the TASKA Hand with other elbows or power systems, please refer to System User Manual for operation.



do more

TASKA[™]


6. Charge

Battery charging

Remove the hand and arm socket from your residual limb before charging the hand. The charging point is a magnetic socket next to the power button.

To charge the hand:

1. Place the magnetic end of the charge cord on top of the charging socket. These will automatically align.
2. Connect the charger to either a wall socket for the Mains Charger, or a vehicle power outlet for the Car Charger which does not exceed 12V. During charging, the charger light will change to red. Once complete, the light will change to green.

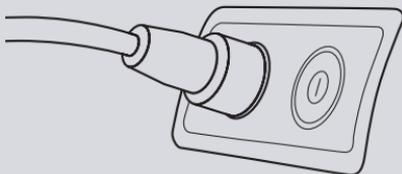

Connecting charging cable to magnetic charging socket on Power Switch.

 Only use the chargers provided by TASKA to charge the TASKA Batteries.

This guide is a quick start use only. Please refer to the **Appropriate Use Guide** for full detailed guidelines on safe use of the TASKA Hand.