




# Quick Start Guide

TASKA Medium (7¾")  
& Large (8¾") Hand V1.1

 **TASKA™**


# 1. Check

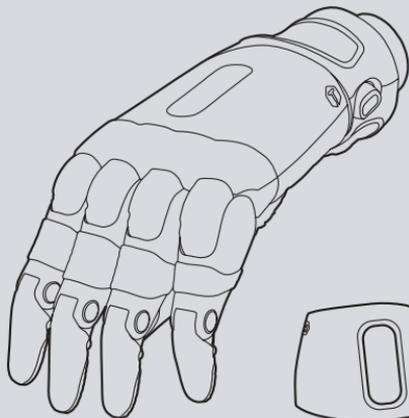


**TASKA Hand\***




**2 x Top Faceplates**

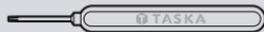


**TASKA Screwdriver**

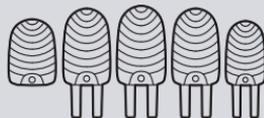


**1 x Thumb & 4 x Finger Grip Tips**

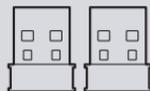

**2 x Bluetooth Adaptors**

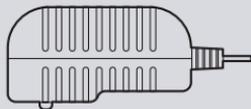

**Mains Charger**

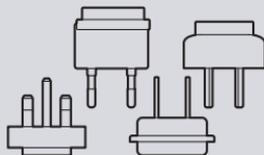

**Universal Plug Kit**



**Car Charger**

 do more

**TASKA™**

\* Image shows Quick Disconnect Wrist fitting.  
Items shown are not to scale.

## 2. Download

### HandCal for users

To be directed to the download page for your TASKA HandCal user software, please enter the link below:


**[taskaprosthetics.com/support](https://taskaprosthetics.com/support)**

For assistance on using the HandCal software, comprehensive guides are available for download on our website.

### HandCal app

A free app is also available for mobile which allows configuration of button and EMG grip sets as well as troubleshooting sensor issues.

To download please visit the app store on your device and search **TASKA HandCal**.

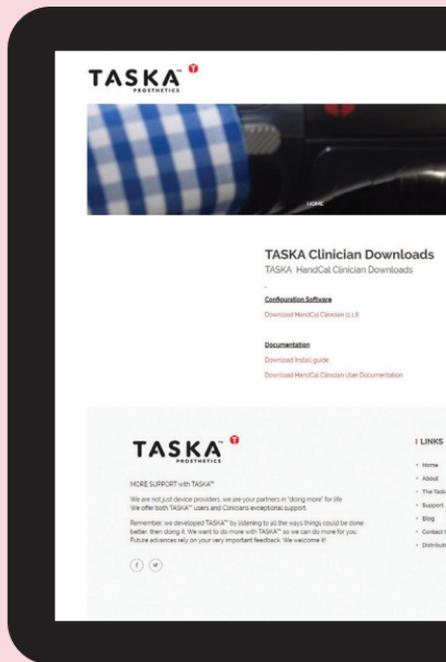

# 3. Control

## Fault light

If this light stays on, there is an internal error. You won't be able to move the hand until you turn it off and on again.

Run the HandCal software to check for problems with your hand. Contact your clinician if this happens regularly.

## EMG functions enabled

Light indicates when all EMG grip triggers or all inputs/EMG sensors are turned on and off.

## Button grip set light

Light flashes momentarily when the first grip in the button group is selected.

## Home grip set

Press this button to toggle between flexi-tool and general grasp. To select pincer precision grip, press the button for one second.

## Button group triggers

Press this button to cycle through your predefined button group grips.

Press for one second to turn Bluetooth on or off.

## Battery light

Light flashes every 10 seconds when the battery reaches 10%. When the battery is flat, the light will stay on and you won't be able to move the hand.

## Bluetooth light

Illuminated when enabled.

## EMG functions

Change this button's function using the Handcal software to turn off EMG sensors or turn off EMG triggers.

Turn sound on/off with one second hold.



## 4. Attach

---

### Connecting the hand


If your TASKA Hand has been supplied with a Quick Disconnect Wrist, it is easy to attach and remove the hand.

#### To attach the hand:

1. Align the wrist connection of the hand with the socket.
2. Push them together. You will hear a click when the hand has locked to the socket.

#### To remove the hand:

1. Rotate the hand and socket in opposite directions and pull the hand gently away from the socket.
2. When the rotation has reached the correct point, the hand will release from the socket.

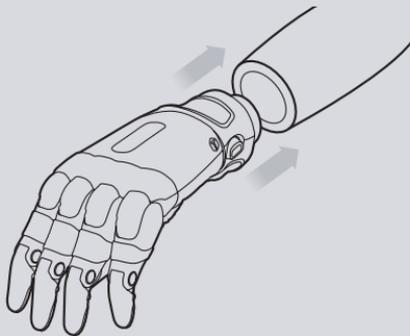

Connecting the Quick Disconnect Wrist.

---

### Low Profile Wrist

If your TASKA Hand has been supplied with a Low Profile Wrist, the hand may only be installed or removed by a trained clinician. **Do not attempt to attach or remove the hand yourself.**


## 5. Power

### Turning the hand on and off

There is a button on the socket that turns your TASKA Hand on and off.

Your hand requires to open fully when being powered on. Ensure there is no restrictions in the movement to ensure the best possible calibration of your hand.

To turn on your hand, press the button once. As the hand turns on, two of the lights on the back of the hand will flash briefly and the fingers will open when the hand is ready.

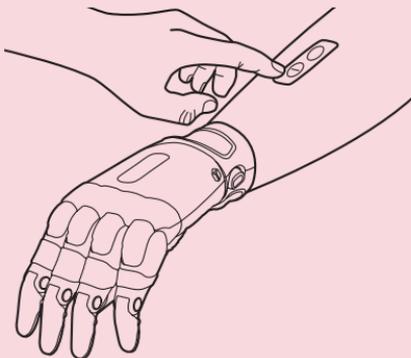

Turning the TASKA Hand on and off using the TASKA Power System.



If you are not using your TASKA Hand with the TASKA Power System, please refer to the Manufacturers Instructions for operation.

 do more

**TASKA™**


## 6. Charge

### Battery charging

Remove the hand and arm socket from your residual limb before charging the hand. The charging point is a magnetic socket next to the power button.

#### To charge the hand:

1. Place the magnetic end of the charge cord on top of the charging socket. These will automatically align.
2. Connect the charger to either a wall socket for the Mains Charger, or a vehicle power outlet for the Car Charger which does not exceed 12V. During charging, the charger light will change to red. Once complete, the light will change to green.

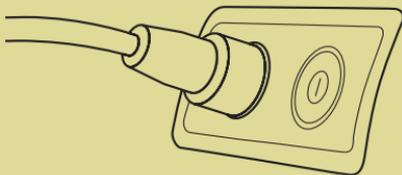

Connecting charging cable to magnetic charging socket on Power Switch.



Only use the chargers provided by TASKA to charge your TASKA Batteries.



do more

**TASKA**

# 7. do more

## Additional resources

As you become more skilled using your TASKA Hand, you will want to further explore what it can do and learn how to use it more for your daily activities.

We provide a range of resources to help you get the most value out of your TASKA Hand:

- Instructional videos.
- Clear, step-by-step training guides.
- News on our social media channels.  
Find us on [@TaskaGlobal](#) and engage with the TASKA community.
- Apps for your mobile device, laptop, or desktop computer for customising your hands operation.

For support and advice on using the TASKA Hand, you should first contact the clinician who worked with you to fit and familiarise you with the hand.

For further support or help in using your TASKA Hand, contact your clinician, or email [support@taskaprosthetics.com](mailto:support@taskaprosthetics.com)

Please refer to the **Appropriate Use Guide** for full detailed guidelines on safe use of the TASKA Hand.